# USER GUIDE.

OrCam **Read.**

 ORCAM | [orcaml.com](http://orcaml.com) |    

OrCam **Read.**

**A SMART PEN THAT READS TEXT FROM ANY SURFACE.**

# List of Languages.

**01. ENGLISH** / 13. French / 25. German  
/ 37. Danish 47. Spanish / 59. Finnish  
/ 69. Japanese / 81. Dutch / 91. Norwegian  
/ 101. swedish

**Get Familiar.** ----- 02

**Get Started.** ----- 03

- Charging.
- Turn on OrCam Read.
- Enter Suspend mode.
- Exit Suspend mode.
- Turn off OrCam Read.
- Setting the Volume.
- Vocal Commands.
- Audio Settings Menu Options.
- Bluetooth Connectivity.
- Mobile App.
- Connect to WiFi and Software Updates.




**OrCam Read Features.**----- 08

- Reading with the Trigger Button.
- Reading Navigation.
- Laser Pointer Modes.
- Set Zone and Time & Telling the Time and Date.
- Smart Reading.

**Terms and Conditions.** ----- 09

**Safety and Warnings.** ----- 10

# Get Familiar.

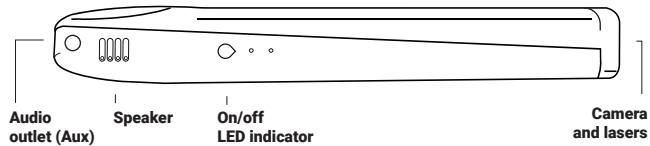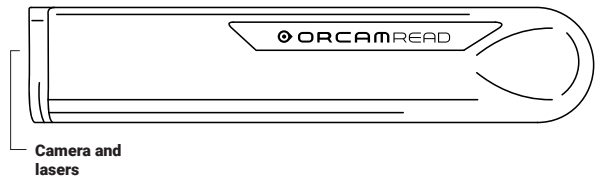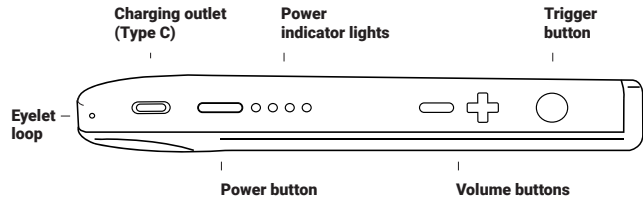## WHAT'S IN THE BOX:

- OrCam Read device
- User Guide
- Charging cable
- Charger box
- Wired earphones
- Cleaning cloth
- Lanyard

# Get Started.

## Online Resources.

You can find more details and information on the OrCam User Area. Visit our website at [www.orcam.com/user-area](http://www.orcam.com/user-area) and enjoy a detailed User Manual, tips, advice, and troubleshooting support.

## Charging.

Before turning on the device for the first time, OrCam recommends charging the device for at least three hours. For all subsequent charges, OrCam Read fully charges in 90-120 minutes.

To charge OrCam Read:

1. Insert the cable into the type C inlet of OrCam Read.
2. Plug the USB end into the charger and then into an electrical socket.

## Turn on or off Your OrCam Read.

- To turn on OrCam Read, press the power button for 2 seconds. When the device wakes up, you will hear, "OrCam Read Version [xx] is ready. Battery is [xx] percent charged".
- To enter the Suspend mode, press the power button once. You will hear a beep and, "Suspending. Press again to shut down", and then, "Suspended" as the device enters Suspend mode.
- To exit the Suspend mode, either press the power button or trigger button once. You will hear a beep and, "Waking up. Battery is [xx] percent charged".

- To turn off OrCam Read, press the power button once. You will hear a beep and, "Suspending. Press again to shut down". Press the power button again. You will hear a beep and, "Shutting down, please wait". As the device shuts down, you will hear, "Power off. Bye bye".

## Setting the Volume.

The volume is set either using a vocal command or manually.

### To increase the volume:

- Rapidly press the trigger button twice and say after the long beep, "Volume up".
- Press the plus ("+") volume button.

### To decrease the volume:

- Rapidly press the trigger button twice and say after the long beep, "Volume down".
- Press the minus ("-") volume button.

## Vocal Commands.

OrCam vocal commands can be used to easily control your OrCam Read, change settings, and receive information.

### To use them:

1. Rapidly press the trigger button twice. You will hear a long beep.
  2. Say the vocal command. You will then hear a chime.
  3. OrCam Read will then tell you the information, change the setting, or ask you for more information in order to change the setting.
- To hear the list of vocal commands, say one of the following commands:

### “Help Tell”

- OrCam Read will list all of the available commands that begin with “Tell”.
- These commands will help you receive general information (such

as the time or date) or information about the device (such as the software version or device serial number).

### “Help Settings”

- OrCam Read will list all of the available commands that will help you modify your device’s settings.

### “Help Smart Reading”

- OrCam Read will list all of the available commands for the Smart Reading feature.

## Audio Settings Menu Options.

Customize your OrCam Read’s settings via the audio settings menu. There are two options for entering the audio settings menu:

- Rapidly press the trigger button twice and say after the long beep, “Enter the user menu”.
  - Press and hold down the power button, press any of the volume buttons, and then release the power button.
- Once you are in the audio settings menu, pressing the trigger button once confirms the selected option; pressing either of the volume buttons browses through the options. Rapidly pressing the trigger button twice either makes the device go back to the main menu or exits the audio settings menu.

## Complete List of Settings Options and Commands.

For the complete list of settings options, OrCam Please commands, and vocal commands, visit the online User Manual on the OrCam website’s User Area.

## Bluetooth Connectivity.

Use Bluetooth to connect your OrCam Read to an external speaker or earphones.

**1.** On the speaker or the earphones:

**A.** Press the Bluetooth pairing button. You will receive an audio response from the speaker or earphones.

**2.** On your OrCam Read:

**A.** Do one of the following options:

- Rapidly press the trigger button twice and say after the long beep, "Connect to the Bluetooth audio device".

- Enter the audio settings menu. Go to Connectivity Settings Menu, then to Bluetooth Settings Menu.

**B.** Select "To connect to a Bluetooth audio device, press the trigger button; for other settings, press the volume buttons". OrCam Read will list all available Bluetooth devices.

**C.** Press the volume buttons to scroll through the Bluetooth

device options. Press the trigger button to select the device to which you want to connect.

**D.** On some Bluetooth speakers or earphones you will hear a "connected" announcement.

**Note:** When connected to a Bluetooth speaker or earphones, the volume level is controlled by the speaker's or earphones' volume buttons only, not by the OrCam Read plus or minus volume buttons.

## Mobile App.

The OrCam Read app allows you to customize your OrCam Read's settings, control reading navigation, and find your device if you misplace it. The app is currently available only to iPhone and iPad users.

**1.** Search and download the OrCam Read app in the App store of your iPhone.

**2.** Turn on your iPhone's Bluetooth mode.

**3.** Follow the onscreen instructions while pairing your device.

## Connect to WiFi and Software Updates.

Connecting your OrCam Read to a WiFi network ensures that your device will install the latest software updates throughout the warranty period. The device must also be connected to the charger throughout the downloading process in order to receive the software updates. Please note that OrCam Read only supports 2.4 Ghz WiFi, not 5 Ghz WiFi.

To connect to your WiFi network:

**1.** On your computer or smartphone:

**A.** Go to [www.orcam.com/wifiset](http://www.orcam.com/wifiset).

**B.** Follow the instructions on the screen to set your WiFi network.

**C.** A QR code will appear on the screen.

**2.** On your OrCam Read:

**A.** Connect OrCam Read to the charger.

**B.** Hold OrCam Read in front of the screen (computer screen about 12-16 in [30-40 cm]; smartphone screen about 6-8 in [15-20 cm]) in order to scan the QR code.

**C.** Read the QR code either using a vocal command or manually.  
- Rapidly press the trigger button twice and say after the long beep, "Read the QR code".

- Point OrCam Read at the OrCam Please command and the QR code on the screen. Press the trigger button.

**D.** OrCam Read will chime and make several beeps as it scans the QR code and connects to the WiFi network. It will announce when it is connected.

**Note:** To check the status of your OrCam Read's connectivity to the internet, rapidly press the trigger button twice and say after the long beep, "Check the internet connectivity".

# OrCam Read Features.

## Reading with the Trigger Button.

**1.** Hold OrCam Read steady, directly facing the text at a distance of about 6-14 inches (15-35 cm) away from the text. Make sure that the long side of the device with all of the buttons is facing upward.

**2.** There are two ways to read with the trigger button.

- Press the trigger button once to take a picture of all text that is present.

- To read more specific text, use the laser pointers.

**A.** Press and hold down the trigger button, which will activate the laser indicator and a border laser frame or mouse laser pointer will be projected onto the text.

**B.** Adjust the border laser frame or mouse laser pointer in order to capture the desired text.

**C.** Let go of the trigger button. You will then hear a camera shutter sound as OrCam Read takes a picture of the text.

**3.** OrCam Read will make several beeps as it processes the text. It will then begin reading the text and chime when finished reading.

**Note:** If your finger is not released before 5 seconds transpire, the camera will not activate.

## **Reading Navigation.**

- To pause or resume reading, press the trigger button twice.
- To skip to the next sentence, press the plus (“+”) volume button once.
- To go back to the previous sentence, press the minus (“-”) volume button once.
- To stop reading, press the trigger button once.
- When OrCam Read is finished reading, it will make a chime sound.

## **Laser Pointer Modes.**

You can switch the laser pointer mode by pressing the plus and minus volume buttons at the same time.

There are two laser pointer modes:

- “Border” for reading framed text.
- “Mouse” to start reading at a specific location.

**Note:** OrCam Read captures the selected text within 5 seconds of pressing and holding the trigger button. If your finger is not released before 5 seconds transpire, the camera will not activate

## **Set Zone and Time & Telling the Time and Date.**

- 1.** To set the internal clock, go to the Time Set page on the website: [www.orcam.com/timeset](http://www.orcam.com/timeset).
- 2.** Press the trigger button to read the “OrCam Please Set Zone and Time” command on the screen.
- 3.** There are two ways to hear the current time:
  - Rapidly press the trigger button twice and say after the long beep, “Tell the time”.

- Use the mouse laser pointer to read the following command:

### **OrCam Please Tell Time**

- 4.** There are two ways to hear the day of the week and the date:
  - Rapidly press the trigger button twice and say after the long beep, “Tell the date”.
  - Use the mouse laser pointer to read the following command:  
**OrCam Please Tell Date**

- 5.** After a few seconds, OrCam Read will tell you the current time and/or date.

# Smart Reading.

This feature enables you to find specific information in any document.

- 1.** Hold the document in front of you with the camera directly facing the text, just as when reading any text with OrCam Read.
- 2.** Rapidly press the trigger button twice and say after the long beep, "Smart Reading".
- 3.** Hold still until you hear the camera shutter sound.
- 4.** Shortly after, the device will announce, "Ready". It is now always listening. There is no need to rapidly press the trigger button twice before every vocal command.
- 5.** For a full list of available Smart Reading vocal commands,

either say, "Help", or visit the online User Manual on the OrCam website's User Area.

- 6.** If you wish to say a new command while the device is reading, press the trigger button once and the device will return to "Always Listening" mode.
- 7.** To exit this feature, either say, "Exit", or press and hold down the trigger button for 5 seconds and then release.

## Terms and Conditions.

Your use of your OrCam device (the "Product") and the mobile app is subject to the Terms and Conditions found at [orc.com/terms-and-conditions](http://orc.com/terms-and-conditions). Your agreement to these Terms and Conditions is a condition to your use of the Product.

Please read the User Guide in full, including the important Safety Instructions, before using your OrCam device. Do not use your OrCam device until you have received in-person or online training by an authorized OrCam Trainer™ or OrCam Training Program.

OrCam can take no responsibility if the Product was used other than in accordance with the instructions in the User Guide. Products should not be relied upon in circumstances that are potentially life- or health-threatening or that could lead to financial loss. OrCam devices should especially never be relied upon to drive vehicles or operate heavy machinery of any kind. OrCam shall not be responsible in the event that Products are used in such situations.

OrCam may receive information from third parties (i.e. Barcodes of Grocery Products in selected countries). OrCam received Barcode information from Barcodes of Grocery Products in Australia and New Zealand from GS1 Australia. OrCam accepts no responsibility for the accuracy of the information obtained from third parties.

## Privacy.

OrCam thinks highly of your privacy and we are committed to protecting and safeguarding our clients' and users' personal information. Please read our Privacy Policy found at <https://www.orcam.com/privacy-policy/>, which describes how we process personal data regarding our clients and users in connection with services provided by us, our website, and the OrCam Read mobile application (the "Services" and the "App", respectively); the types of personal data we collect and process; to what purposes we use and process your personal data; how your personal data is protected; and with whom your personal data may be shared.

It is important for us to enable you to understand our privacy policy and your rights in connection with the personal data we hold about you. You should also be aware that we use cookies and similar technologies to collect and save information when you visit our website, available at [www.orcam.com](http://www.orcam.com), to ensure that we give you the best possible experience. To learn more about the use of such technologies, please see our Cookie Policy at <https://www.orcam.com/en/cookies-policy/>.

If you feel that a concern you have in connection with your privacy has not been met by the Privacy Statement, please contact us at: [privacy@orc.com](mailto:privacy@orc.com) and let us know.

If you choose to download the App or use, interact with, register, or provide data to us in connection with the Services, you explicitly agree to the use of such data in accordance with this Privacy Statement. You may not use the Services or submit any data to us if you do not agree to any of the terms hereunder. This Privacy Statement shall not be construed in any manner to derogate from our Terms of Conditions or any other agreement made by and between OrCam and you.

## Warranty.

Customer acknowledges that, other than as explicitly stated herein, OrCam does not make any representations or warranties to Customer regarding the Product, its functions or any images it produces.

The OrCam end-user limited product warranty (the "Limited Warranty") covers the owner of an OrCam product against defects in material or workmanship at the time of its original purchase, for a subsequent period of one (1) year, and for any additional applicable legal warranty period. Original proof of purchase/delivery is required from the owner in order to obtain this coverage.

OrCam will repair or replace, at its discretion, any product that is determined by OrCam to be covered under this Limited Warranty. OrCam will not be responsible for conditions arising as a result of intentional damage or misuse of the product. The owner is responsible for the costs of shipping the product to the location specified by OrCam.

This Limited Warranty is the sole and exclusive warranty applicable to OrCam products. No third party has the authority to make any representation, warranty, or agreement on behalf of OrCam with respect to OrCam products. No warranty of any kind or nature is made by OrCam beyond those expressly stated herein.

In no event shall OrCam be liable for any special, collateral, indirect, punitive, incidental, consequential, or exemplary damages in connection with or arising out of the use of the product. In no event shall OrCam's liability exceed the price paid for the product.

Without limiting any provisions limiting liability under this agreement, the limited warranty set forth herein will not apply, and the owner will reimburse OrCam for any costs and expenses incurred in the event that:

- (i) the product has been used other than in accordance with the product documentation and terms of use or other written operating instructions provided by OrCam
- (ii) the product has been subject to tampering, misuse, negligence, or accident
- (iii) the product or parts identification labels have been removed or the product has been otherwise altered
- (iv) the product has been opened, modified, repaired, serviced, maintained, or altered by someone other than an authorized OrCam representative

(v) the product has been combined with software, hardware, or other equipment not supplied by OrCam or not approved in writing by an authorized OrCam representative

(vi) the product has been intentionally damaged or exposed to water, heat, or high voltage

(vii) the product has been used by a person who has not completed training by an authorized OrCam Trainer™

Any parts replaced pursuant to this warranty will be under warranty for the remaining time of the original warranty period or for ninety (90) days from the date of repair, whichever is longer.

To register your device for warranty, please visit the warranty registration website: <https://discover.orcam.com/warranty>.

If you believe that your product needs warranty support, contact [support@orcaml.com](mailto:support@orcaml.com). You will be responsible for shipping costs. However, if it is determined that the warranty request is valid, OrCam will pay the total cost of shipping the repaired or replaced product.

ORCAM DISCLAIMS ANY AND ALL WARRANTIES, WHETHER EXPRESSED OR IMPLIED, INCLUDING WITHOUT LIMITATION, THE IMPLIED WARRANTIES OF NON-INFRINGEMENT OF THIRD-PARTY RIGHTS, MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. ORCAM DOES NOT WARRANT THAT THE PRODUCT WILL BE ERROR FREE OR "BUG" FREE, WILL OPERATE WITHOUT INTERRUPTION, THAT ANY IMAGES OR SOUNDS PRODUCED BY THE PRODUCT WILL BE ACCURATE OR THAT THE OPERATION OF THE PRODUCT WILL BE SECURE AND HEREBY DISCLAIMS ANY AND ALL LIABILITY ON ACCOUNT THEREOF. THIS SECTION SHALL BE ENFORCEABLE TO THE FULLEST EXTENT ALLOWED BY APPLICABLE LAW.

### **Refund Policy.**

You can return your undamaged device for a refund within the period required by local law or as agreed between you and OrCam at the time of purchase. Training fees are non-refundable. Please contact your local representative or OrCam Support to initiate a return process.

When you return the OrCam device, please follow OrCam Support instructions carefully. The device must be in the same condition that you received it and undamaged in any way. Your refund and how it is issued may differ based on how the device was purchased, its condition, and how long you've had the device.

OrCam will process your return only after the carrier has received your device and delivered to OrCam. It usually takes up to three weeks for OrCam to receive (for international shipping involving local customs it may take longer) and process your return. Once OrCam issues your refund, it may

take additional time for your funds to be made available in your account by your financial institution. For credit card payments, it may take 5 to 10 business days.

If you have any questions, please feel free to contact the OrCam Support team.

### **FCC Compliance.**

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician.
- This device complies with Part 15 of the FCC Rules.

Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference.
- (2) This device must accept any interference received, including interference that may cause undesired operation.

### **FCC Warning.**

Modifications not expressly approved by the manufacturer could void the user authority to operate the equipment under FCC Rules.

NOTE: THE MANUFACTURER IS NOT RESPONSIBLE FOR ANY RADIO OR TV INTERFERENCE CAUSED BY UNAUTHORIZED MODIFICATIONS TO THIS EQUIPMENT. SUCH MODIFICATIONS COULD VOID THE USER'S AUTHORITY TO OPERATE THE EQUIPMENT.

Waste Electrical and Electronic  
Equipment Directive Compliant  
Contact OrCam for proper disposal.

Restriction of Hazardous  
Substances Directive Compliant  
(materials and components)

FCC ID:  
2AAWI-READ

CE Compliant

Refer to User Instruction  
Guide and Manual

## Safety and Warnings.

Before using the OrCam device, read these precautions carefully to avoid dangerous situations and to ensure ideal performance of your device. Handle your OrCam device with care. It can be damaged if dropped, burned, punctured, or crushed. Do not open the outer casing of the device.

The OrCam device is designed to work in ambient temperatures between 32°F and 104°F (0°C and 40°C) and in normal conditions (Relative Humidity of 10% to 90% RH and Atmospheric Pressure of up to 2000m above sea level (700hPa)). The device can be damaged if used outside of this range. Store and transport in a cool, dry place (32°F to 104°F (0°C to 40°C) and Relative Humidity of 5% to 90% and atmospheric pressure of 700hPa to 1060hPa)). The OrCam device is water resistant but not waterproof. This means the device can withstand light rain or drizzle for short periods of time, although it is not recommended. Do not use the OrCam device in heavy rain, at the beach or swimming pool, or in or near a bathtub or shower. Do not submerge the OrCam device in any liquid. The OrCam device meets the standards of IP Code: IP22.

Should you have a problem operating the OrCam device, if the case or lens is cracked, or if you believe there is something wrong with the device, please contact support and do not attempt to repair it by yourself. Only qualified OrCam personnel may repair or service your device. Unauthorized repairs may void your warranty.

Do not attempt to replace the battery; you may damage it, which could cause overheating and even injury. The battery is designed to last for two years of average use (one charging cycle per day on average) after which it may need to be replaced. Use only the charger supplied with your OrCam device. Using another charger may void your warranty and could cause severe injury.

Do not force a connector into a port or apply excessive pressure to a button. This may cause damage that is not covered under the warranty. If a connector does not insert easily, check for obstructions in the port and verify that you have the correct connector for that port. Do not combine with software, hardware, or other equipment not supplied by OrCam. Exposing the device to magnetic fields may damage the unit or erase your data.

When using the OrCam device or charging the battery, it is normal for the device to get warm. If the unit becomes excessively hot, leaks, or gives off a burning smell, turn the device off immediately and contact OrCam Support.

If you have specific questions about use alongside an internal pacemaker and/or defibrillator, please consult your medical professional. Do not use the OrCam device to read medication labeling.

## Cleaning.

Clean your device if you feel it is dirty or transferred between users. Never touch the camera lens with your finger. Clean the lens only with the supplied microfiber cloth and eyeglass cleaning fluid. Do not use any abrasive materials. Do not expose the device to chemicals, such as household cleaners, aerosol sprays, solvents, alcohol, or ammonia. If you experience an allergic reaction to any of the elements of the OrCam device, contact a medical professional.

Max output of laser radiation : 0.371 mW
Emitted wavelength : 635 nm
Name of the standard : IEC 60825-1

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